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13 Proper use

The lift must only be used to lift and lower disabled persons into a vehicle as specified in the technical description.

Any other use is regarded as improper. The manufacturer / supplier is not liable for any damage resulting from such improper use. The user alone bears the risk.

Proper use of the lift involves observing and following the operating instructions and honouring the inspections- and maintenance conditions.

The lift has been built state-of-the art and in accordance with the recognised regulations concerning safety. However, the use of the lift involves an element of danger to life and limb for the user or a third party; or impairment of the lift functioning or damage to other property.

The lift may only be used in proper working in accordance with the safety regulations and in full awareness of the danger involved. The operation instructions must be precisely followed. Especially faults which impair the safety have to be repaired as soon as possible.

The user must take sure, that

- Only trained staff or staff under instruction may work with the lift.
- The lift is only operated in proper working order.
- Staff instructed to work with the lift must have read these operating instructions and especially the chapter on working before starting work.
- Before trial run, any safety devices which have to be removed during maintenance or repair work are replaced.
- It is not allowed to carry out any alterations or changes on the lift for reasons of safety, without the manufacturer permission.
- The operator may only start up the lift if his is certain that no person / no pet is standing in the danger area.
- In the case of any changes in the operation behaviour which could impair safety the lift must be stopped and remove the motive.
- Switch off the lift before maintenance or repair work and make sure that no one can switch on the lift during this works.
- After maintenance or repair work make sure that all safety devices replaced before start the operation.
- Follow the rules of traffic during the operation in the street.

AMF-Bruns GmbH & Co. KG | Hauptstraße 101 | 26689 Apen
Telefon +49 (0) 44 89 / 72 72 -22 | Fax +49 (0) 44 89 / 62 45
hubmatik@amf-bruns.de

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Easy Access Teleskop-Plattform (EATP)

Easy Access Splitt Plattform (EASP)

The Linearlifts are made up the following assemblies:

1. hoisting gear
2. lifting platform
3. hydraulic device
4. electrical device

Changeable devices
(-none-)

14.1 Function

Linearlift Easy Access (EA)

The Linearlift Easy Access (EA) exists of two lifting arms and one standard lifting platform. The operation ensue by a hand switch for the functions „up“, „down“, „in“, „out“ as well as „stow position“.

Linearlift Easy Access with split-platform (EASP)

The Linearlift with split-platform exists of two lifting arms and one lifting platform, which will flap in and out during fold-in and fold-out operation. In neutral position in the vehicle this lifting platform needs space as like as a standard platform, but the viewing out will be free. The operation ensue by a hand switch for the functions „up“, „down“, „in“, „out“ as well as „stow position“.

Linearlift Easy Access with telescopic-platform (EATP)

The Linearlift with telescopic-platform exists of two lifting arms and one lifting platform, which will make a telescopic move during fold-in and fold-out operation. In neutral position in the vehicle this lifting platform needs not so much space as a standard platform, but in folded-out position there is the same space on the platform. The operation ensue by a hand switch for the functions „up“, „down“, „in“, „out“ as well as „stow position“.

The hydraulic aggregate is located inside of the vehicle. It is optimal covered against dirt, water and weather conditions.

The bearings are maintenancefree and need no grease. The drive ensue electrohydraulic with 12V main voltage.

AMF-Bruns GmbH & Co. KG | Hauptstraße 101 | 26689 Apen
Telefon +49 (0) 44 89 / 72 72 -22 | Fax +49 (0) 44 89 / 62 45
hubmatik@amf-bruns.de

www.amf-bruns.de

Reg.-Nr.: Q1 0105027

22 BEDIENUNGSANLEITUNG / OPERATING INSTRUCTION / MANUAL

15 Safety regulations

Safety regulations (VBG 14) in extract:

§45 operating instructions

Note during operation of lifting devices the operating instructions.

§46 appropriation

(3) Lifting devices which stands in the traffic or rise into the traffic space have to protect in a right manner.

§47 manipulation and behaviour during the operation

(1) Lifting devices don't have to load over the allowed loading.

(3) Get on/down lifting devices only by the allowed entrance.

(5) While all moves of the lifting device the staff have to observe that there not endanger himself and other persons (pets).

(6) It's not allowed to stay unnecessary in/on the moving area of lifting devices.

Further not allowed is:

1. to stay under the lifting platform,
2. the using as a lift bridge for works if the lifting device is not designed for it.

§51 Abandon operation

Forced lifting devices have to secured after abandon operation against unallowed using.

Warning about remaining risks remaining risks are not to shut out, therefore:

Caution! Assembly work have to carry out only by qualified staff. The relevant safety instructions must be observed.

16 Operation

Qualification, training, incorporation of the staff

Only trained members of staff or staff under the instruction of a qualified person may operate the lift. We don't assume any liability for any damage resulting from improper operation.

Notes on safety regulations, warning of danger

The following should be read carefully and attentively, in order to avoid operation faults and personal accidents!

The lift operator is responsible for observing the following safety functions- and regulations and should have instructions for this purpose!

Particularly important informations in this operating instructions have been marked by the following denominations:

Remark! A remark give important informations to make a procedure comprehensible.

Caution! Under the denomination attention special procedures have been listed which must be observed in order to avoid damages to the lift or to the vehicle!

Warning! A warning indicate special measures which must be observed absolutely in order to avoid injuries!

AMF-Bruns GmbH & Co. KG | Hauptstraße 101 | 26689 Apen
Telefon +49 (0) 44 89 / 72 72 -22 | Fax +49 (0) 44 89 / 62 45
hubmatik@amf-bruns.de

www.amf-bruns.de

Req.-Nr.: Q1 0105027

Pushbutton „Down“

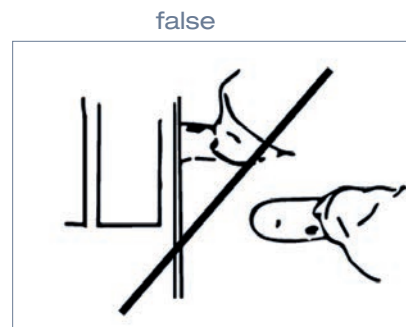
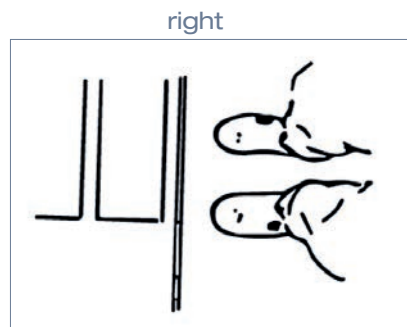
Pushbutton „Up“

1. Lower the platform by push on the pushbutton „down“, till the platform will be lying on the ground and the automatic barrier will be folded-down.

Caution! Take care, that there is no person in the swivel-out area of the lift. Don't get with the feet into the lowering area of the platform or reach with parts of your body into the clearance between the brackets!

Remark!

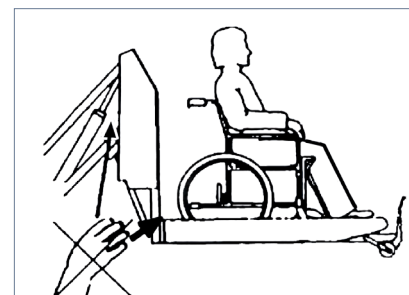
2. Wheelchair occupant move on the platform. We commend to move the platform backward.#

Caution! Driving on the platform by electrical wheelchair is only admissible when its motor has been turned off! The transport of companions on the platform is not allowed!

Drive in the center of the platform and never block the barrier with the wheels of the wheelchair!

3. Raising the platform by pushing on the pushbutton „up“ until it will be reached at the upper edge of the vehicle floor.

Caution! Don't reach with partsof your body into the clearance between the brackets!
Take care, that there is no person in the swivel area of the lift!

4. By pushing on the pushbutton „up“ once again the lift will be swivelling into the vehicle.

Caution! Make sure, that there is nobody in range of the lift! To prevent damage of the hydraulic, release the pushbutton „up“ if the lifte reached the stow position!

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18 Getting-out

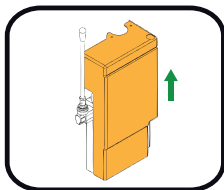
1. Swivel-out the platform by push on the pushbutton „down“. Lowering the platform by pushing on the pushbutton „down“ until it will be reached at the upper edge of the vehicle floor. Wheelchair occupant move the platform.

Warning! Don't drive with the wheelchair against the barrier. Drive in the center of the platform! Driving on the platform by electrical wheelchair is only admissible when motor has been turned off. The transport of companions on the platform is not allowed.

2. Lowering the platform by pushing on the pushbutton „down“ until it will be reached at the ground. The barrier fold down automatically in this position.
3. Bring in the lift into neutral position by pushing on the pushbutton „up“ the lift will be swivelling into the vehicle.

Caution! Make sure, that there is nobody in range of the lift! To prevent damage of the hydraulic, release the pushbutton „up“ if the lift reached the stow position!

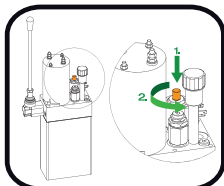
19 Emergency operation manual

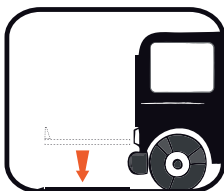
For emergency operation only, please go to workshop immediately afterwards.

Remove the cap of the power pack. Then do either operate emergency lowering or lifting as described below:

Emergency lowering

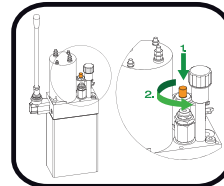
Press the lowering valve's red knurled head screw down, and rotate it anti-clockwise at the same time until it snaps up (lowering valve is open).

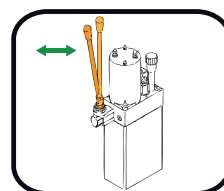
Lift platform is now lowering automatically.

Emergency lifting

Press the lowering valve's red knurled head screw down, and rotate it clockwise at the same time until it snaps up (lowering valve is closed).

Raise platform manually on handset, or in case of a power failure by manual pump, to stow position, and stow lift into the vehicle.

AMF-Bruns GmbH & Co. KG | Hauptstraße 101 | 26689 Apen
Telefon +49 (0) 44 89 / 72 72 -22 | Fax +49 (0) 44 89 / 62 45
hubmatik@amf-bruns.de

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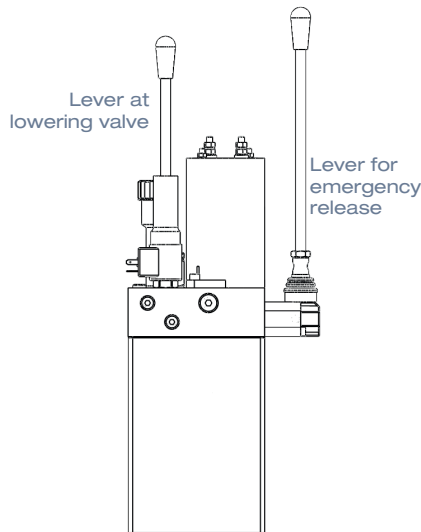
21 Emergency release with lever

Emergency release

Press the lever on the top of the lowering valve and turn it at the same time to the left until it locked in place. Platform will lower automatically.

Emergency lifting

In case of previous emergency release, press the lever on the top of the lowering valve and turn it at the same time to the right until it locked in place. Afterwards the lifting platform can be lifted with the handpump lever into the stowed position.

20 Service and maintenance

Qualification, training, incorporation of the staff

Faults may only be repaired by trained members of staff or staff under the instruction of a qualified person may operate the Back-in-Box-Lift. We don't assume any liability for any damage resulting from improper operation.

Safety regulations, warning of danger

Wear protective clothing and use properly functioning safety devices when carrying out servicing or maintenance work. The instructions of the safety officer in charge must be followed. Especially important is sufficient cleanliness and regular checking of the protective devices and safety switches.

Caution! Only carrying out servicing or maintenance works with the Lift switched off. Secure the Lift against unintentionally switching on before starting work!

Method and interval of service and maintenance works

The Lift have to be checked orderly. Unsuccessful and improper servicing causes malfunctions.

Orderly checks by a competent person

After the first commissioning the lift have to be checked in distances of one year by a competent person. Prepare the lift, if necessary clean it, so that the check can be carry out with the regulations.

The orderly check is essential a visible- and function check. The check is comprising:

- the condition of the components and the ascertainment in case of alterations.
- the completeness and effectiveness of the safety devices.
- the completeness of the inspection book.

26 BEDIENUNGSANLEITUNG / OPERATING INSTRUCTION / MANUAL

22 Malfunctions and their removal

Fault	Possible cause	Possible remedey
Linearlift doesn't swivel out	1. Handswitch is defect. 2. Battery is empty.	1. Check connections- if necessary change the switch 2. Charge Battery
Linearlift doesn't sink	1. Handswitch is defect. 2. Battery is empty. 3. Lowering valve is defekt	1. Check connections- if necessary change the switch 2. Charge Battery 3. Change lowering valve.
Linearlift doesn't raise	1. Handswitch is defect. 2. Battery is empty. 3. Contactor is defekt	1. Check connections- if necessary change the switch 2. Charge Battery 3. Change contactor..
Linearlift doesn't swivel in	1. Handswitch is defect. 2. Battery is empty.	1. Check connections- if necessary change the switch 2. Charge Battery
Linearlift sinks automatically slowly	1. Lowering valve does not close faultlessly	1. Change lowering valve.

Above mentioned malfunctions can give only a short overview about the multitude possible malfunctions. Because of the Lift design it's not too difficult for the maintenance staff to identify and repair any other malfunctions. The following points should always be considered:

- Switch off the electricity
- Search and repair the cause with consideration.
- Don't cause an additional damage through force

AMF-Bruns GmbH & Co. KG | Hauptstraße 101 | 26689 Apen
Telefon +49 (0) 44 89 / 72 72 -22 | Fax +49 (0) 44 89 / 62 45
hubmatik@amf-bruns.de

www.amf-bruns.de

Req.-Nr.: Q1 0105027

27 BEDIENUNGSANLEITUNG / OPERATING INSTRUCTION / MANUAL

23 Transportation

Qualification of the staff

Only trained or properly instructed staff may load, unload and transport the lift. The minimum legal age must be observed. We accept no liability for damage resulting from improper transportation.

Notes of safety regulations, warning of danger

The lift must be carefully fixed and secured to the lifting equipment for loading and unloading to avoid any dangerous situations. Only suitable lifting equipment and load suspension devices with sufficient carrying capacity; and which are in perfect working order may be used.

Caution! It is not permitted to work or stand below suspended loads.

24 Assembly - General notes

General notes

The following steps must be taken before starting assembly:

- Check that the machine parts to be assembled are complete with the help of the shipping notes and assembly plans.
- Visually inspection of the machine parts for any damage.
- Check that the place of assembly has a sufficient carrying capacity and spatial assembly possibilities.
- Clarify what measures have to be taken in emergency situations, such as accidents or fires, etc. with a member of staff.

Caution!

- Wear safety and protective clothing when carrying out assembly work and only use safety devices in perfect working order. The relevant safety instructions must be observed.
- Only qualified personal have to connecting up the lift.

Staff qualifikations

Only trained members of staff or staff under the instruction of a qualified person may assemble the lift. We do not assume any liability for any damage resulting from improper assembly.

Notes on safety regulations, warning of danger

When carrying out assembly work always wear safety and protective clothing. Only safety devices which are in perfect working order may be used. The relevant safety instructions must be observed.

25 Assembly instructions

- 1.) Prepare vehicle floor.
- 2.) Arrange the lift into the vehicle as shown in assembly drawing and drill lift mounting holes (d 11mm) for bolts M10x30-8.8. (See drawing)
- 3.) Mounting the bolts M10x30-8.8 (ISO 4014) with plates (DIN 125) and nuts (ISO 4032).
- 4.) Elektrical wiring as shown in diagram.

AMF-Bruns GmbH & Co. KG | Hauptstraße 101 | 26689 Apen
Telefon +49 (0) 44 89 / 72 72 -22 | Fax +49 (0) 44 89 / 62 45
hubmatik@amf-bruns.de

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26 Technical Datas

It is allowed to transport

One disabled person who is sitting in a wheelchair and whose wheelchair in weight / size is not stronger / larger as the lifting power / size of the lift / platform.

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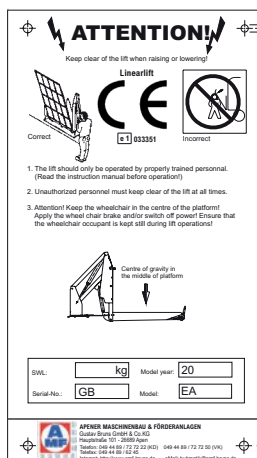
30 Lable Markering

Masine plate	:	number plate
Attaching place	:	Lifting platform

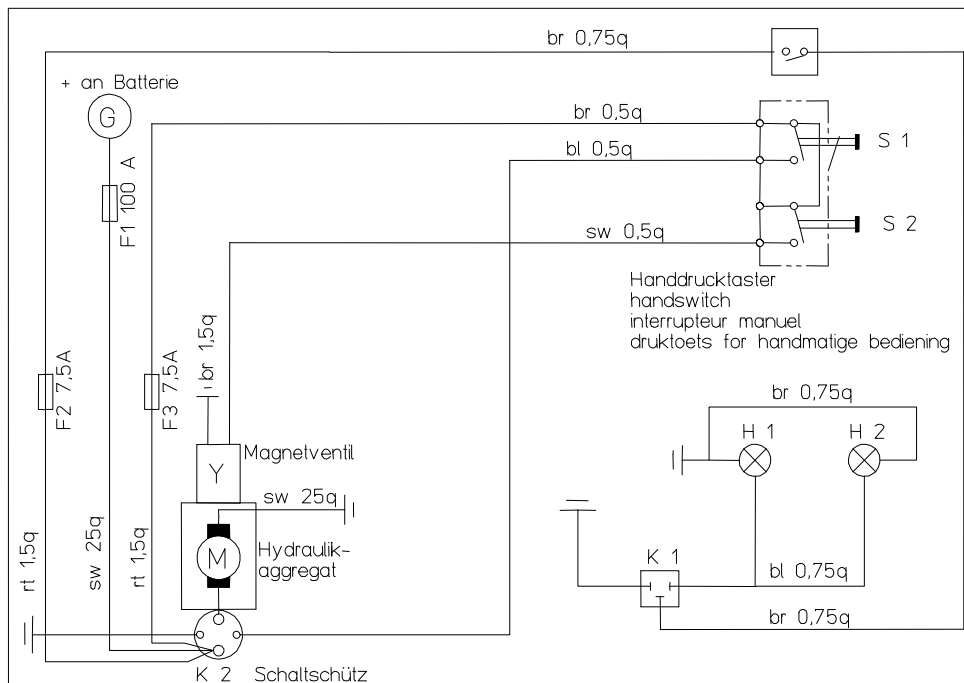
AMF-Brunns GmbH & Co. KG | Hauptstraße 101 | 26689 Apen
 Telefon +49 (0) 44 89 / 72 72 -22 | Fax +49 (0) 44 89 / 62 45
 hubmatik@amf-bruns.de

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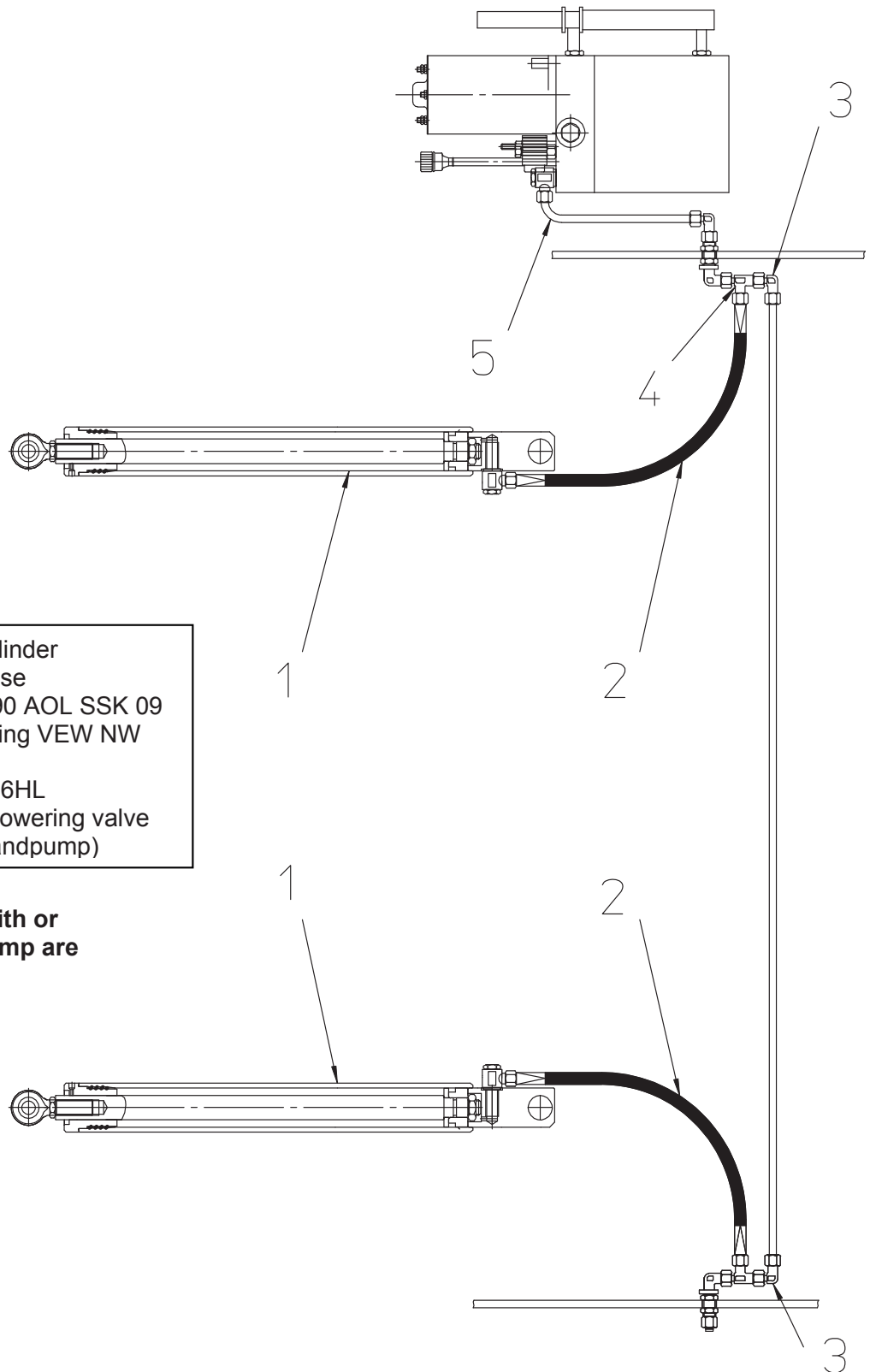
www.amf-bruns.de

Req.-Nr.: Q1 0105027

- 1) Hydraulic cylinder
- 2) Hydraulic hose
PHD 106x390 AOL SSK 09
- 3) Angle screwing VEW NW
- 4) T-screwing
T VEL NW 06HL
- 5) Emergency lowering valve
(only with handpump)

Note:
Hydraulicplan with or
without hand pump are
indentical!

AMF-Brunns GmbH & Co. KG | Hauptstraße 101 | 26689 Apen
Telefon +49 (0) 44 89 / 72 72 -22 | Fax +49 (0) 44 89 / 62 45
hubmatik@amf-bruns.de

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www.amf-bruns.de

Reg.-Nr.: Q1 0105027

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70 Directions for ordering spare parts

Please state:

Lift type

Year of manufacture

Serial no.

Item no.

Orders can be placed via telefax on 0049 4489 6245 or by email to manfred.meiners@amf-bruns.de.

70.1 Warranty

Warranty claims are only regulated if accepted and approved by AMF-Brunns beforehand.

70.2 Exchange delivery

In case of an exchange delivery (e.g. warranty claim):

1. Please return the parts that need exchanging as quickly and cost-savingly as possible, otherwise we will have to charge them to your account.
2. Please send the covering letter separately to the following address:

AMF-Brunns GmbH & Co. KG
Abt. Hubmatik Aftersales
FAO Mr. Manfred Meiners
 Hauptstraße 101
 26689 Apen / Germany

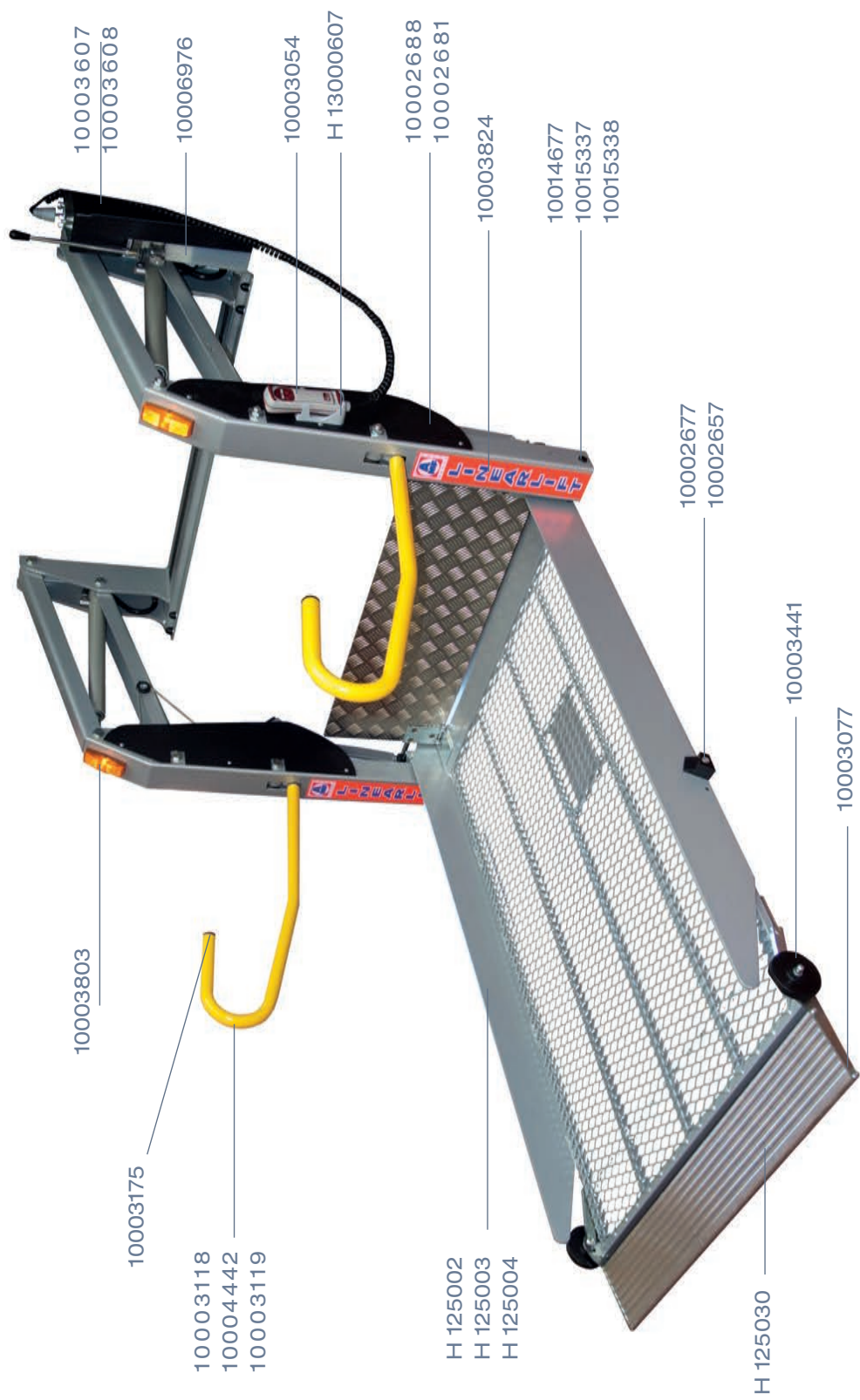
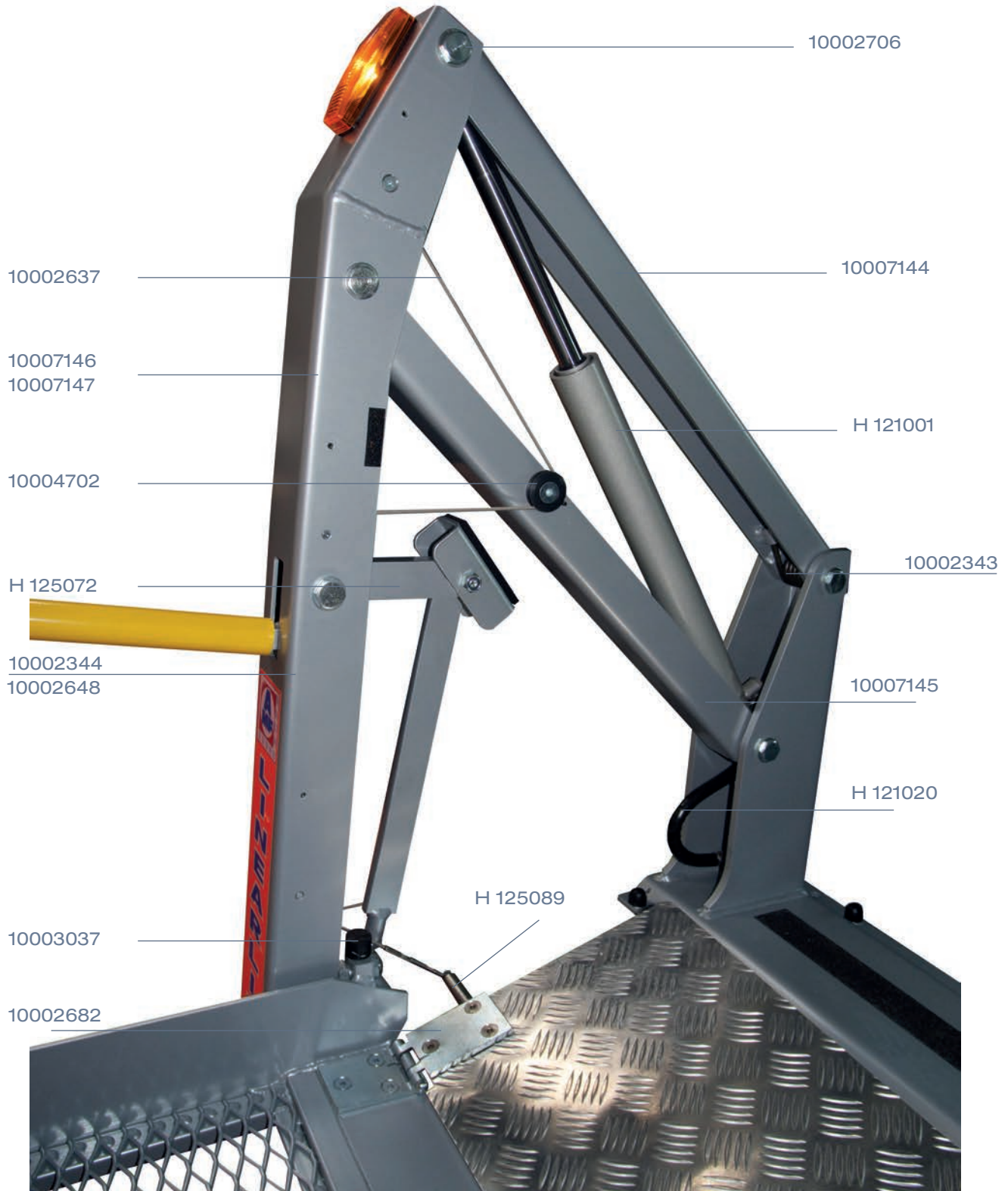
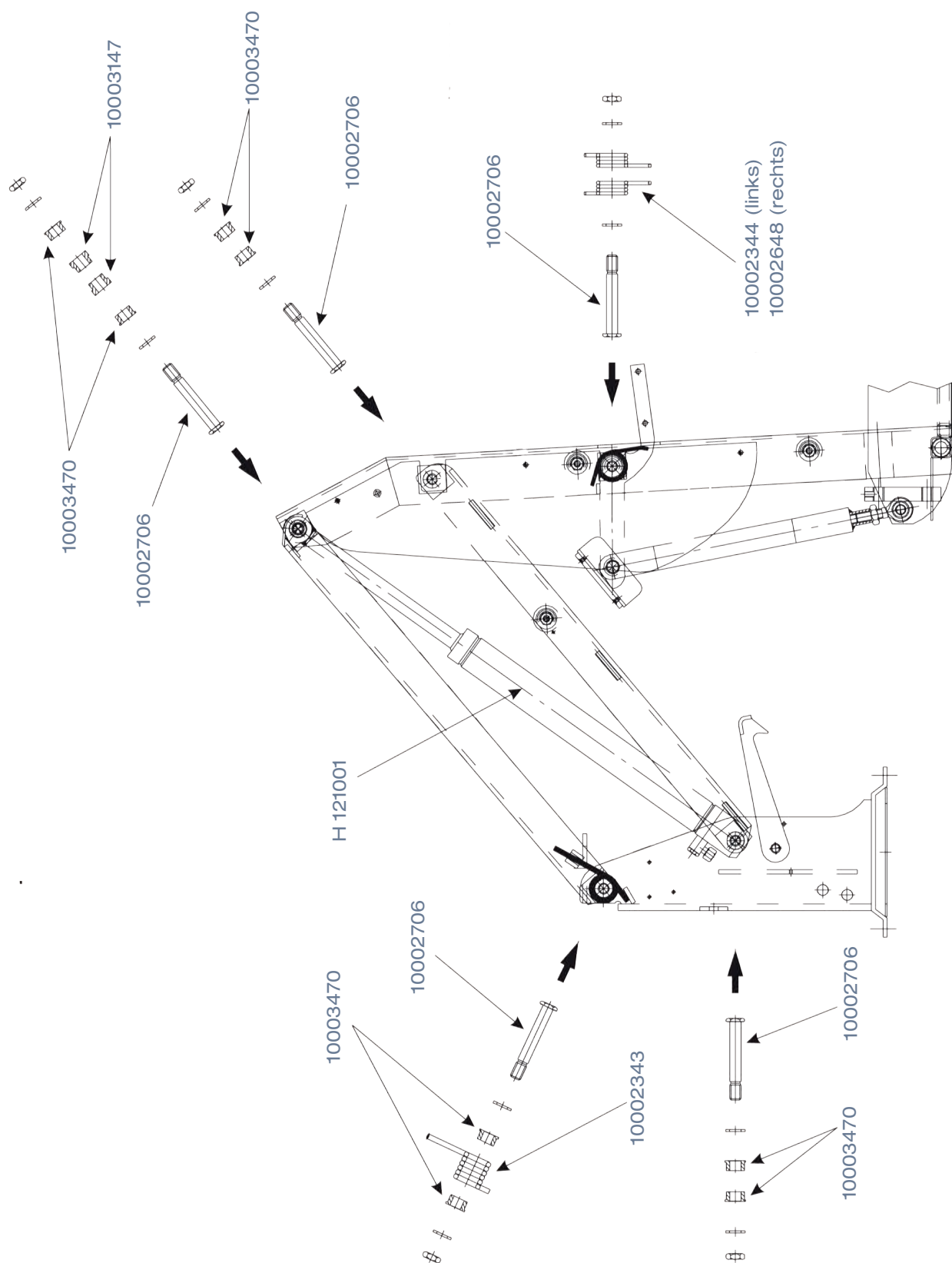
AMF-Brunns GmbH & Co. KG | Hauptstraße 101 | 26689 Apen
 Telefon +49 (0) 44 89 / 72 72 -22 | Fax +49 (0) 44 89 / 62 45
hubmatik@amf-bruns.de

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